

Healthy Safety Respirator [Masks](#) Dust Pollen Allergens Flu Germs Masks



Features:

Human body engineering 3D stereo design

Multilayer non-woven fabric and meltblown, total 4 layers

Aluminum belt hold shape



## Applicable scope

can be used to protect non oily particles,reduce the odor of some organic compounds.

Non oily particulates:dust(mineral property,metallic dust fiber dust)pollen and haze.

Which can be use to protect environment air pollution fine particle PM 2.5.

Can't be suitable for:

[Respirators](#) do not produce oxygen,not in oxygen-deficient environment.

remember:Can only be used for pollutants, the wrong use will lead to disease or even death.



## Steps for usage

step 1

Open the [mask](#), nose clip out.Both hands were pulled side of the band, so that the nose folder above the mask.

step 2

Put the mask against the chin.

step 3

Pull the band to the ear.

step 4

Place your hands in the middle of the metal nose clip.From the middle of the nose by the status quo, until close to the bridge of the nose.

step 5

Into the work area□Feel the air leak from the nose, should be re-adjust the nose clip.If you feel no air overflow can enter the work area.

