

## New design [neck mask](#) VT103 wholesale in China

- Unique shape design allows low profile and good fit
- Enhanced comfort due to easily adjustable nose bridge
- High density and high degradation mask for PM2.5




### Features

Human body engineering 3D stereo design

Multilayer non-woven fabric and meltblown, total 4 layers

Aluminum belt hold shape

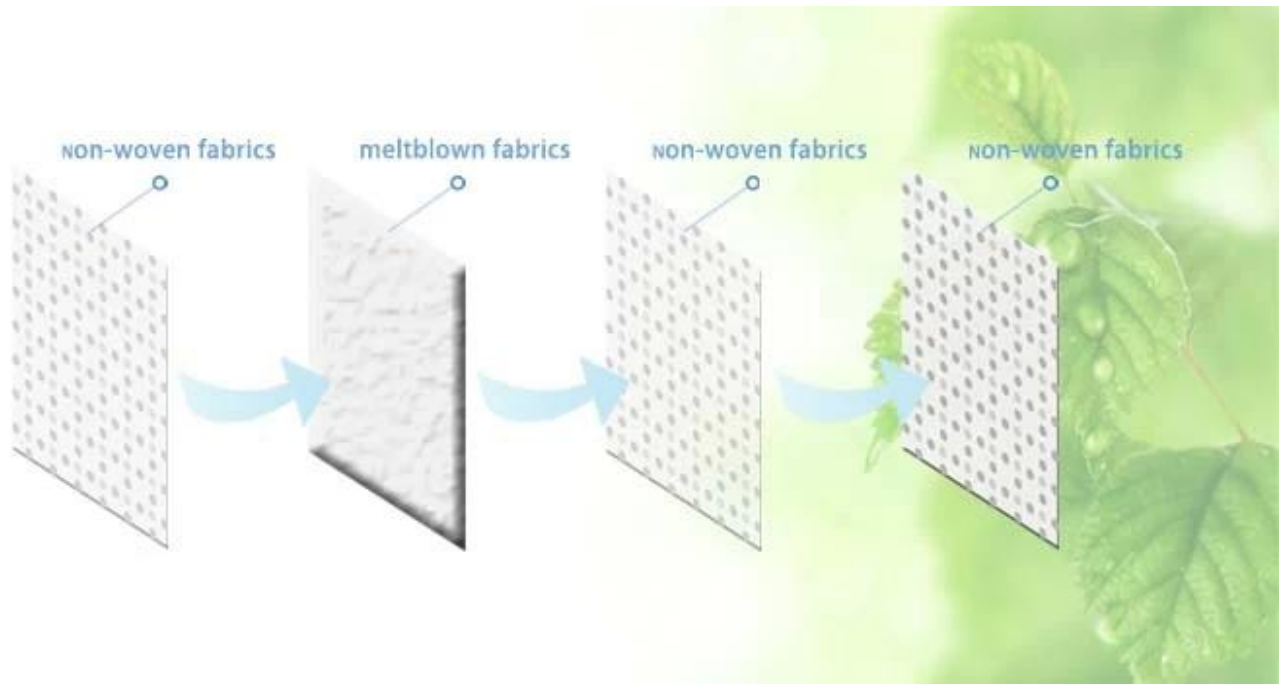

## USE LIMITATIONS AND WARNINGS

- 1.The wearer must first be trained in the proper use and fit tested before using this respirator.
- 2.Do not use this [respirator](#) when concentrations of contaminants are immediately dangerous to life or health.
- 3.Leave the contaminated area immediately if breathing becomes difficult,or dizziness or other distress occurs.
- 4.Discard and replace respirator if it becomes damaged or breathing resistance becomes excessive.
- 5.Inspect respirator before each use to ensure it is in good working comdition.Examine all the respirator parts for signs of damage including the two straps,noseclip and staples.
- 6.Do not alter,wash,abuse or misuse this respirator.
- 7.Do not use with beards or other facial hair or other conditions that prevent a good seal between the face and the sealing edge of the respirator.

### **Warning:**

**This respirator helps protect against certain particles,misuse can result in sickness or death.**

## Guard your family and your health

Prevention of UV rays      Cool and comfortable

Suitable for a variety of occasions



### Steps for usage

Perform hand hygiene before putting on [mask](#).

Choose the correct size of the mask and ensure there are no defects.

1. Hold the VT103 in your palm with the nosepiece at fingertips, hang ear straps freely below hand.

2. Position the 103v under your chin with the nosepiece up.

While holding the VT103 in place, pull the two straps over each ear.

3. And then pull the hoop to another strap, make it in place on the neck.

4. If you have long hair, the strap must be positioned beneath your hair.

Position the VT103 low on your nose.

5. Using both hands, mold the nosepiece to the shape of your nose by pushing downward and outward.

While moving your fingertips down both sides of the nosepiece.

6. Perform a fit check by placing both hands completely over the mask.

Be careful not to disturb the position and exhale sharply.

If air leaks around your nose, adjust the nosepiece as described in step 5.

If air leaks at the mask edges, adjust the straps back along the sides of your head.

Perform fit check again if an adjustment is made.

### Remember:

Wear the right sized mask

Fit check is mandatory for all users of VT103 mask and must be performed every time upon wearing.

Perform hand hygiene after putting on mask.

