

[Infrared forehead Thermometer](#) for Body, Objects & Ambient Temperature

1 Second measurement



Features

The Medexpro [JT004 thermometer](#), further to its reliability, accuracy, and easy handling, allows gentle and non-invasive measurement of body temperature without touching the patient.


1 Seconds

Fast and gentle measurement in only 3 seconds.

3 in 1

Multifunctional

Measurement of forehead, objects & ambient temperature in seconds.

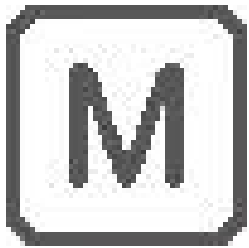

30 Data Memory

[Automatic storage](#) of the last 30 measurement values with date and time.



Specification:

Touch-free measurement in only 1 sec.

Multifunctional (body, object and ambient temperature)
Tracking light - for easy use also at night

Clinically tested and recommended by doctors
Silent Glow™ technology and fever alarm

Large illuminated display with date and time
30 data memory (stored with date/time)


Sound control (On/Off)
Optional: Storage case or storage bag

Mod. No.: NC 150
Size: 141.1 x 43.3 x 36.9 mm
Weight: 90g (with battery), 67g (w/o battery)
Measuring range: 0 - 100.0 °C
Measurement accuracy: ± 0.2 °C between 36.0 and 39.0 °C
Battery: 2 x 1.5 V alkaline batteries, size AAA

User instructions:

Step 1:

You can press Mode BUTTON to switch the mode between Forehead mode and Object mode. Press the Power Button or the START Button, after showing of full screen, the thermometer is now ready for a new measurement. If there is an abnormal, the Er1 or Er2 appears.

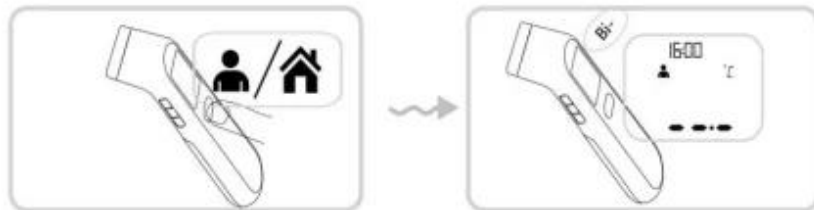

Step 2:

The temporal artery is connected to the heart via the carotid artery. So it is designed to measure the skin surface around temporal artery, a major artery of the head. Aim the thermometer at the center of the forehead with a distance of 2 to 3 cm and then press the START BUTTON. Note: Do not remove the thermometer from the forehead before hearing the final beep.



Step 3:

Read the temperature on the display.

1. If the temperature reading is below 37.8°C (100.0°F), a happy face will be displayed next to the reading.
2. If the temperature reading is 37.8°C(100.0°F) or above, a bad face will be displayed next to the reading.



Step 4:

To prolong battery life, press the Power Button to turn unit off after testing is complete. If no action is taken, the unit will automatically shut off after around 1 minutes.

Store the thermometer in its protective case.