


Features:

Human body engineering 3D stereo design

Multilayer [non-woven](#) fabric and meltblown, total 4 layers

Aluminum belt hold shape

4 layers protection

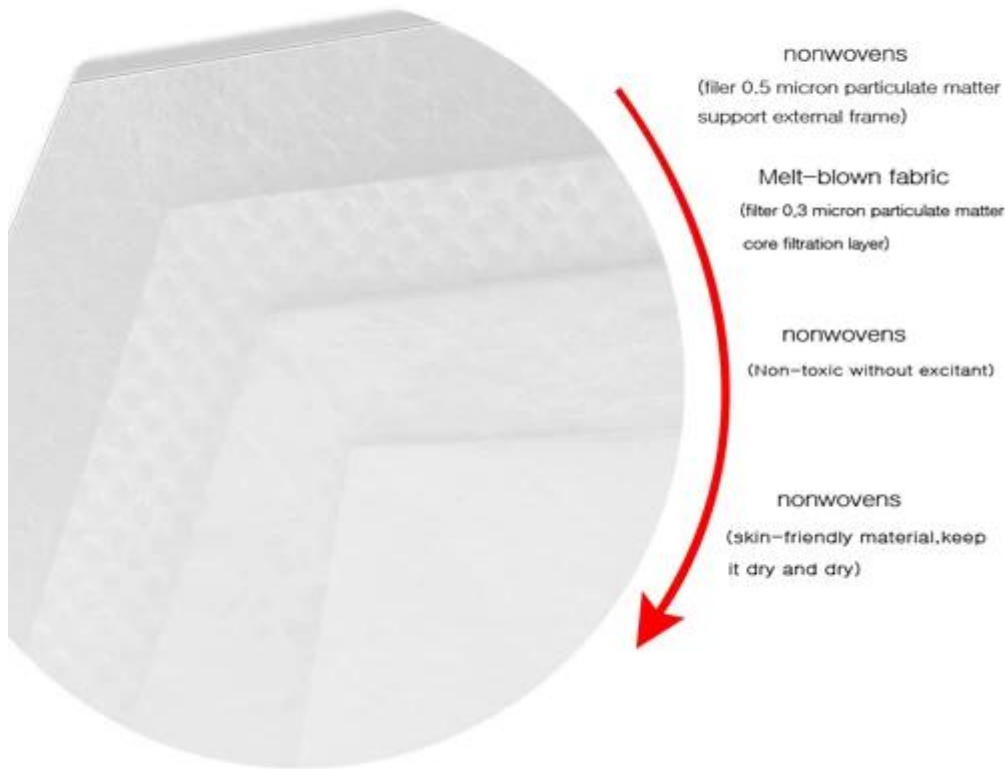

Steps for usage

step 1

Open the [mask](#), nose clip out. Both hands were pulled side of the band, so that the nose folder above the mask.

step 2

Put the mask against the chin.

step 3

Pull the band to the ear.

step 4

Place your hands in the middle of the metal nose clip. From the middle of the nose by the status quo, until close to the bridge of the nose.

step 5

Into the work area □ Feel the air leak from the nose, should be re-adjust the nose clip. If you feel no air overflow can enter the work area.



Usage range

can be used to protect non oily particles,reduce the odor of some oganic compounds.

Non oily particulates:dust(mineral property,metallic dust fiber dust)pollen and haze

Which can be use to protect environment air pollution fine particle PM 2.5

Can't be suitable for:

[Respirators](#) do not produce oxygen,not in oxygen-deficient environment

Guard your family and your health

Prevention of UV rays Cool and comfortable

Suitable for a variety of occasions

