

Medexpro provides top quality [masks](#) which can be printed [logo](#).If you are interested ,pls enquiry me .

Our [masks](#) are all 3D design and there are six kinds.



Customers items

The custom product itself belongs to the particularity.After confirming the customized sample,the product cannot be changed.It is suggested to confirm the information with the customer service before production.

The filtration effect of non oil particulate matter was at least 96.7%

customizable

logo+ diagram

Custom items

The custom product itself belongs to the particularity. After confirming the customized sample, the product cannot be changed. It is suggested to confirm the information with the customer service before production.

Steps for usage

Step 1: Select the correct respirator size which you were fitted.

Step 2: Place your palm through the loops of your respirator and cup it over your face.

Step 3: Position the respirator to cover the nose and chin.

Step 4: With one hand holding the respirator, use the other hand to place the first strap onto the crown of the head (above the ears), then place the second strap onto the back of the neck.

Step 5: Using two fingers of both hands, press down the metal piece according to the contour of your nose bridge. (NEVER PINCH THE METAL PIECE!)

Step 6: With both hands over the respirator, exhale strongly to check for leakage around it.

Step 7: Leakage is evident when your hair moves during the strong exhale. If wearing glasses, fogging of the glasses indicate leakage.

Step 8: If air leak is noted, adjust respirator, straps, and nose piece before testing again.

