

Medexpro provides top quality masks which can be printed [logo](#).If you are interested ,pls enquiry me .

Our [masks](#) are all 3D design and there are six kinds.




Professional manufacturer of [mask](#) with enough in stock.

### Customers items

The custom product itself belongs to the particularity.After confirming the customized sample,the product cannot be changed.It is suggested to confirm the information with the customer service

before production.

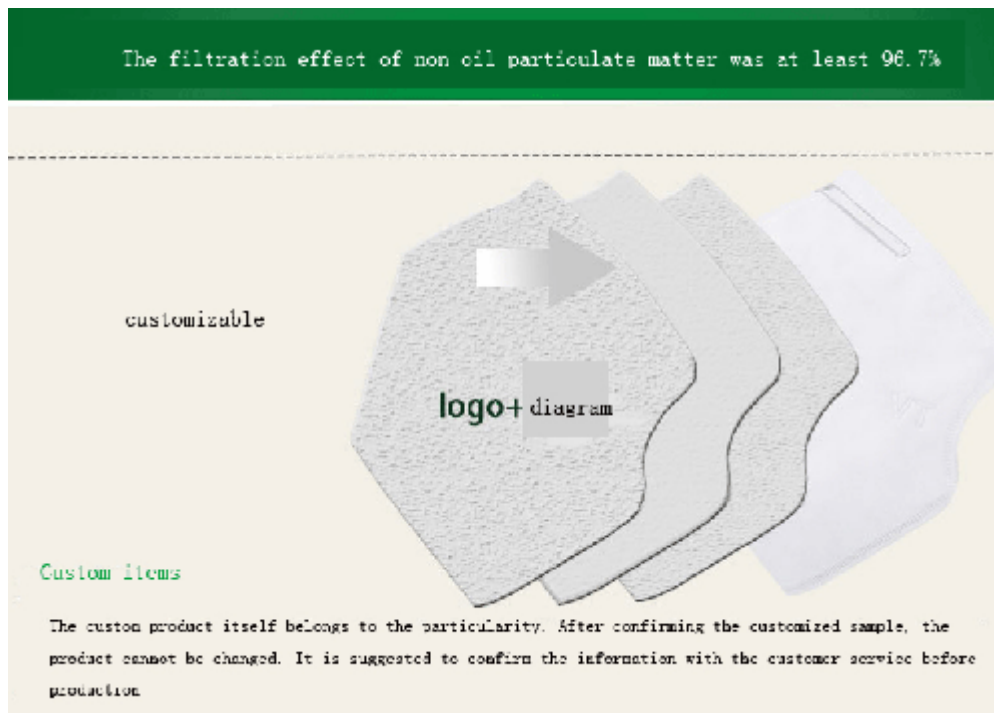

## Steps for usage

### step 1

Open the [mask](#), nose clip out. Both hands were pulled side of the band, so that the nose folder above the mask.

### step 2

Put the mask against the chin.

### step 3

Pull the band to the ear.

### step 4

Place your hands in the middle of the metal nose clip. From the middle of the nose by the status quo, until close to the bridge of the nose.

### step 5

Into the work area. Feel the air leak from the nose, should be re-adjust the nose clip. If you feel no air overflow can enter the work area.

